

Using the NovoFine® Autocover® needle

Attaching the NovoFine® Autocover® needle to your pen device



Step 1:

Take a new needle. Tear off the paper tab



Step 2:

Push the needle straight onto the pen. Turn until it is on tight



Step 3:

Pull off the outer needle cap



Step 4:

Always check the flow before you begin to inject. This helps to ensure that you get your full dose of medicine



Step 5:


Inject your medicine by following the guidelines from your doctor or nurse, and the Instructions for Use that comes with NovoFine® Autocover® and your pen



Step 6:

After injecting, the safety lock indicator will turn red to indicate that the shield is now locked

Inject into the right part of the skin

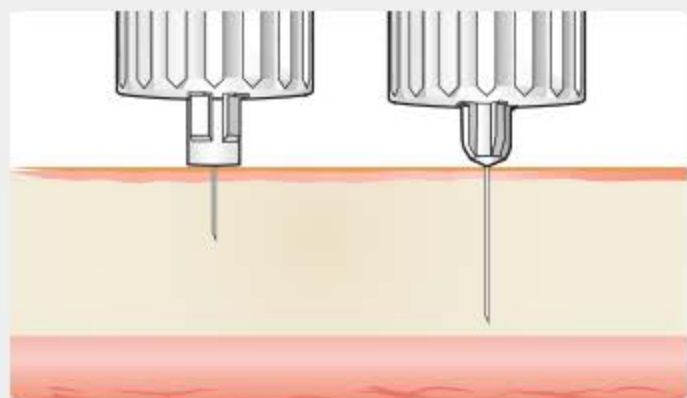

- The medicine must be injected into the fatty layer of the skin called the subcutaneous layer
 - The depth of the subcutaneous layer is different for every person and at different parts of the body
- The length of the needle could influence which injection technique you should use to ensure your medicine is correctly injected
 - Your doctor or nurse will be able to tell you which technique is best for you, and it is best to follow this guidance

Correct needle disposal

To reduce the risk of injury to others, pen needles should be disposed of in a puncture-resistant container, such as a "sharps container", according to local regulations. Check local recommendations for discarding medical waste.

